


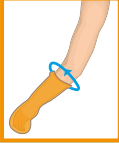

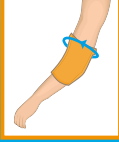



Leg and PICC line/Isolated Wound products are now available through your Doctor.

all our sizes are measured in cms

	Sizes			
	Short Leg	S = 30	M = 40	L = 50
	Medium Leg	S = 30	M = 40	L = 50
	Long Leg	S = 50	M = 70	L = 90
	Short Arm	S = 30	M = 40	L = 50
	Medium Arm	S = 30	M = 40	L = 50
	PICC lines	S = 40 / 40	M = 50 / 40	
	Isolated wound	S = 40 / 40	M = 50 / 40	

other sizes available on request

Prices from £17-£25

[www.buddycover.com](http://www.buddycover.com)

t. office: +44 (0)2920 844582

t. UK Freephone: 0800 2700072

 @buddycover  buddycover

 **buddy**<sup>®</sup>  
dry limb covers

Leg & Arm waterproof wound dressing and cast protectors




Swim



Bath



Shower

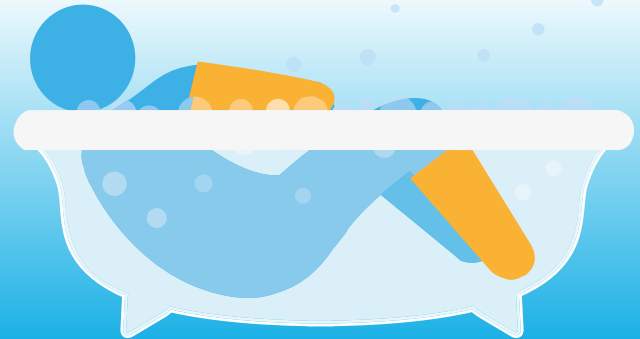

Re-usable



Latex-free

To order visit:

[www.buddycover.com](http://www.buddycover.com)

e. [sales@buddycover.com](mailto:sales@buddycover.com)

t. office: +44 (0)2920 844582

t. UK Freephone: 0800 2700072




Measure in cms around your limb, approx 4cms above/below the cast, dressing or affected area. Then choose the nearest size for you.

# How to use your buddy®



Gently stretch open the seal. Slip your **buddy®** over your limb, pull up gently.

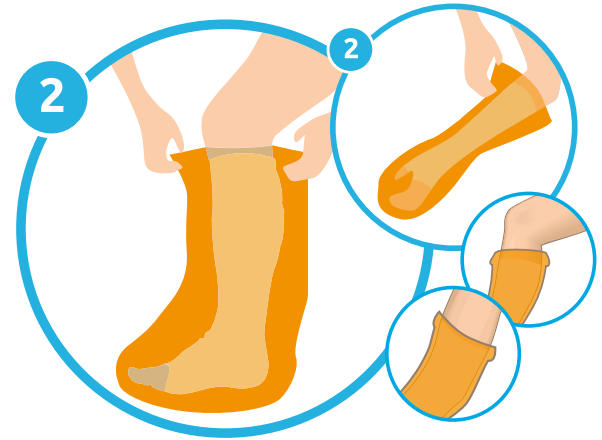

Pull the corners down slightly creating a smooth seal around the limb (this at both ends on the picc line isolated wound cover). **You are ready to shower!**



**For swimming and bathing:**  
Gently pull the corner to open the seal as you lower your limb into the water. Let go of the seal just before you fully submerge your limb.



You can swim, bath and shower with confidence. Your **buddy®** won't let water in.

- Avoid sharp objects whilst wearing/handling your buddy product
- Supervise children with buddy products at all times. This is not a toy.
- Do NOT use bleach, to clean - rinse and leave to dry naturally
- These products can be used inside out.